Lensy Care 7

DE

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EN

All-in-One solution for cleaning, disinfecting, storing and hydrating all types of soft contact lenses.

Contains vegetable stem cell extracts and is enriched with hyaluronic acid.

IMPORTANT DIRECTIONS FOR USE AND SAFETY INFORMATION. PLEASE READ CAREFULLY AND KEEP FOR FUTURE REFERENCE. IMPROPER CONTACT LENS CARE CAN LEAD TO SERIOUS EYE DAMAGE.

DIRECTIONS FOR USE

Before handling the contact lenses, wash and rinse your hands thoroughly and dry them with a clean towel.

Open the lens case.

Take the contact lens and place it in the palm of your hand.

Apply a few drops of solution to each side of the contact lens.

Gently rub the lens in a circular motion for a few seconds.

Rinse the lens again thoroughly with the multipurpose solution.

Place the lens in the lens case and fill it with solution.

Keep the lenses in the solution for at least 4 hours or overnight.

Before using the lenses, rinse them again with the solution.

Throw out the used solution from the lens case.

Rinse the case with the solution and let it dry. NEVER USE TAP WATER

TO RINSE THE CONTACT LENSES.

If the contact lenses remain in the solution for more than 7 days, the cleaning process must be repeated prior to wearing them.

CONTRAINDICATIONS (Reasons not to use the product)

Do not use if you are sensitive or allergic to any of the ingredients in this

product.

PRECAUTIONS

Do not use the product after expiry date.

Make sure the dropper tip is sealed before first use. If already opened,

do not use.

Do not use the solution if the seal around the bottle is not intact (broken,

missing or tampered with).

Close bottle after use.

ONCE BOTTLE IS OPEN, USE THE SOLUTION WITHIN 60 DAYS.

Make sure the dropper tip of the bottle never touches any surface, since

this may contaminate the solution.

Replace lens case with every new bottle.

Store between 5° and 30°C!

Consult your eye care professional before changing solutions.

WARNINGS

Always use fresh solution. Never re-use the solution.

Never transfer the solution to another container.

Never dilute or mix with other solutions.

DO NOT USE TAP WATER TO RINSE THE CONTACT LENSES, only use a

saline solution or a multipurpose solution.

KEEP OUT OF REACH OF CHILDREN.

DO NOT SWALLOW!

Before taking any medicines, consult your eye care professional or

ophthalmologist.

SIDE EFFECTS

If any reactions such as inflammation, persistent itching or irritation or

other eye problems occur, remove the contact lenses and stop using the

product. Contact your eye care professional.

INGREDIENTS

PolyHexaMethyleneBiguanide 0,0002%, Echinacea Plus 0,001%, GPI

0,012%, Hydroxyethylcellulose 0,015%, Poloxamer 407 0,02%,

Hyaluronic acid, Sodium chloride, Citric acid, Tribasic sodium citrate,

Ethylene diamine tetra acetic acid 0,01%, Sodium hydroxide anhydrous,

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